


Calprotectin+FIT DOs and DON'Ts

- ✓ **DO** give the **patient** the FIT sample collection device **AND** calprotectin sample pot (blue/brown universal container) if both tests have been requested
- ✓ **DO** remind **patients** to collect the FIT sample and calprotectin sample from the **same bowel movement**
- ✓ **DO** remind **patients** to label the FIT sample collection device with their name and sample collection date and **complete the consent form** if they would like to participate in the research
- ✓ **DO** return the calprotectin and FIT samples plus consent form to the biochemistry laboratory **together** with the Pathology/ICE request form



You can put everything in the specimen bag.

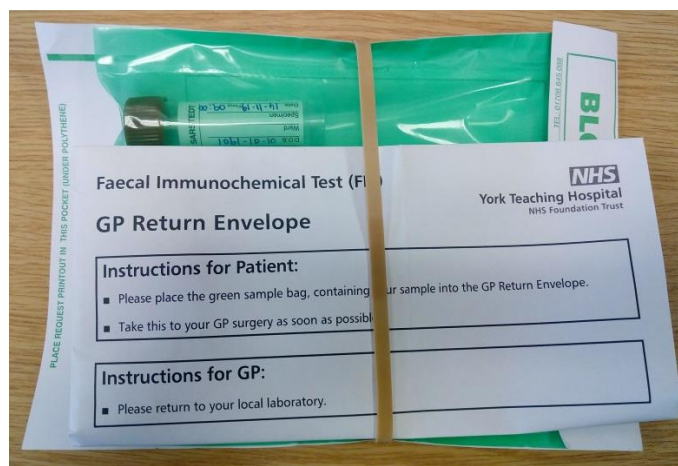

You can secure the specimen bag and GP return envelope with an elastic band.

- ✗ **DON'T** supply FIT collection pack without Pathology/ICE request form.
- ✗ **DON'T** reuse ICE request forms.
- ✗ **DON'T** send calprotectin and FIT samples to the laboratory separately.

Please contact your local biochemistry lab or the Duty Biochemist (01904 72 6366) with questions about FIT collection pack. Orders for more collection packs should be made using the normal pathology consumables request form